

Instruction Manual

PICC Puncture Pad WARNING

Opening the puncture pad



1) Turn the film of the pad.



2) Flip the puncture pad upside down and push down on the corners to empty onto a tray.



■WARNING■
Do not attempt to pull the pad directly out of the plastic container.

⚠ WARNING

- Do not open the package until you are using the product.
- We recommended using these pads within one session. The aqueous material in the product will cause the pad to shrink after it is removed from its packaging.
- Keep the product away from high temperature, high humidity and direct sunlight.



- After it has been used, the puncture pad may be stored again by placing it back into its plastic case with the transparent seal and keeping it in a sealed bag. This method will preserve the product size and quality for a short period of time.

■ Main Office and Factories (World Wide)

 **KYOTO KAGAKU co.,LTD**

WEB•www.kyotogagaku.com
E-MAIL•rw-kyoto@kyotokagaku.co.jp

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan
TEL: +81-75-605-2510
FAX: + 81-75-605-2519

■ Americas

 **KKAmerica**
Kyoto Kagaku America Inc.

WEB•www.kkamerica-inc.com
E-MAIL•info@kkamerica-inc.com

3109 Lomita Boulevard, Torrance, CA 90505-5108, USA
TEL: +1-310-325-8860
(Toll-free in North America: 877-648-8195)
FAX: +1-310-325-8867